

## Users manual for Evac Aid

1. To inflate the Evac Aid open air bottle valve



Safety valve for automatic pressure control, also used for emptying the Evac Aid



2. Place the person in the stretcher



3. Place safety strap under the arms and lock over the chest



4. Place blue marked straps between the knees and lock. Also used for fixating the legs in a 90 degree angle for short stretcher



5. Strap for fixating the legs together. Place straps over the legs and thru the d-ring and lock



6. Strap for fixating the body and arms



7. Use the transport straps for moving the Evac on floors, terrain and stairs. If the stairs are steep we recommend two persons for evacuation. This to avoid risk of overturning the stretcher.



8. After using the Evac fill the air bottle with compressed air 200 bar pressure